2023 – 2024

MEDICAL**ALPRO**GRAMM

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For over 30 years, ALPRO is one of the most innovative companies in the field of infection prevention. Our certified and environmentally friendly products are not only characterized by maximum effectiveness, they also cover the entire range of infection control in the clinic; from hand disinfection, surface and instrument disinfection to the removal of biofilm and its prevention in dental water lines of dental treatment units. ALPRO is one of the world's leading manufacturers of cleaning, disinfection and maintenance products for dental suction systems.

ALPRO attaches great importance to environmental friendliness, aldehyde- and phenol-free products are therefore a matter of course for us. These include the products MinutenSpray-classic/MinutenWipes variants for quick surface disinfection, which have been proven in clinical use for many years, as well as the universally applicable products BIB forte eco (burr and instrument bath) for the disinfection and cleaning of all instruments, or the innovative, gentle alcohol-, aldehyde- and QAC-free surface disinfection PlastiSept eco/Wipes variants for all plastic, synthetic and metal surfaces. Sustainability is also a priority in our technology of the company headquarters, e.g. through the use of solar power and geothermal energy. Here, ALPRO lives up to its social responsibility for emission-free production.

Through constant research and further development of our established products, we are also able to offer flexible and individual solutions for complex issues such as process water hygiene. With BRS® (Biofilm-Removing-System) and Alpron/Bilpron, it is possible to meet the requirements of DIN EN ISO 16954:2015 as well as to achieve drinking water quality in dental treatment units. In order to maintain this standard, which is necessary for the dental clinic, we can provide two different procedural water decontamination systems for connection to the dental treatment unit, Weekend-System II and ALPRO-BCS (BottleCareSystem).

Another highlight of our product range are the cleaning and disinfection solutions of the WL-line for manual reprocessing of handpieces and turbines. The device-friendly solutions comply with the highest cleaning (DIN EN ISO 15883) and disinfection requirements and have also been tested and approved by a wide range of manufacturers.

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Cleaning, disinfection and care of hands

AlproSept-HD	7
AlproSept-HDE	9
CREMANA®-wash	10
CREMANA® Creme-Lotion	11
CREMANA®-derm protect	12

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Cleaning and disinfection of surfaces

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MinutenSpray-classic	16
MinutenWipes 30/MinutenWipes Jumbo 30	17
MinutenWipes 50/MinutenWipes Jumbo 50	18
MinutenWipes Maxi	19

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Cleaning and disinfection for special applications

AlproJet-D	44
AlproJet-DD	45
AlproJet-W	46
AlproCleaner	47
PrintoSept-ID	48
AlproSol	49
TarClean	50
SteriCleaner	51

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Decontamination of procedural water

Alpron	53
Bilpron	54
Biotest	55
ALPRO-BCS StarterSet (BottleCareSystem)	56
BC-San 100	57
Weekend-System II StarterSet	58
i3 ONE Sterilfilter	59
i3 FOUR Sterilfilter	60

Zubehör/Accessories	61 – 63
----------------------------	----------------

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Hygienic hand disinfection (30 seconds)

Application notes

Standard rubbing method for the hygienic hand disinfection according to EN 1500

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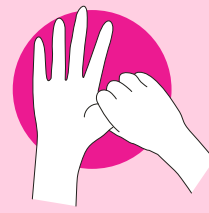
Surgical hand disinfection (3 minutes)

Application notes

Standard rubbing method for surgical hand disinfection according to EN 12791

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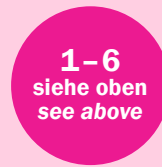
For hygienic hand disinfection rub AlproSept HDE on the hands for 30 seconds.

Spectrum of activity

- **bactericidal** incl. MRSA and suitable for hospitalism prophylaxis
- **yeastcidal**
- **virucidal** active against all enveloped "blood-borne" viruses, e.g. HBV, HCV, HIV, influenza and corona viruses and non-enveloped adeno viruses

Active ingredient basis

Ethanol, propan-2-ol, skin care substances, PHB-Ester.

Biocidal products

Always read the label and product information before use.

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Mild hand-washing lotion with selected active substances. Even in case of frequent hand-washing, the protection substance contained prevents the skin from drying out and keeps it soft and smooth.

Field of application

Ideal for the use in all medical fields that require frequent hand washing, e.g. medical and dental practices, hospitals, etc.

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Cream for intensive skin protection and care for particularly sensitive and stressed skin. CREMANA® Creme-Lotion is easy to rub in, absorbs quickly and does not grease. Suitable for all skin types.

Application

After the cleaning/decontamination of the hands (breaks/end of work) put a cherry stone sized quantity on the back of the hand and rub in thoroughly.

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*Greasing skin protection and barrier cream with special skin care substances. For application at work in case of frequent contact with water based products.
Ideal for skin protection when wearing gloves.*

Application

Apply on washed and dried hands. For the sufficient protection of the hands put a cherry stone sized quantity on the back of the hand and disperse to the palm of the hand.

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- PZN (für 300 ml): 11857502
- PZN (für 1 L): 03645335

Rinsing solution ready for use for application in mouth rinsing devices and ultrasonic devices. Alprox serves for cleaning and care of the mouth cavity as well as of the gingival pockets. Alprox reduces the vitality of periodontitis bacteria, removes plaque and protects against tartar. Alprox does not lead to discolourations of the teeth. Alprox has no negative effect on the action of detergents and disinfectants in the aspiration system.

Active ingredient basis

Modern combination of biguanide and tosylchloramide.

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Alcohol-free wet wipes for the non protein fixing cleaning and decontamination of surfaces of medical devices and intact skin. The bacteriostatic and fungistatic acting CleanWipes in „cotton touch“ quality, remove easily and carefully organic and inorganic soiling from medical devices, hands and intact skin. CleanWipes are dermatologically tested.

Active ingredient basis

EDTA, parabens,
biguanides in glycolic-aqueous solution.

Efficacy/Certification

- cleaning/decontaminating
- dermatologically approved

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REF 4154

REF 4602

dispenser box with 70 wipes
carton with 12 refill bags
wall holder (round dispenser boxes),
for JumboWipes 30/50

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Alcoholic solution ready for use for the quick cleaning and disinfection of surfaces of devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment. Approved amongst others by Dentsply Sirona and Planmeca.

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- **virucidal** (EN 14476, EN 16777) 2 min.
against non-enveloped adeno and noro viruses as well as all enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses

Test criteria: without soiling, room temperature*

- **virucidal** (RKI/DVV) 30 sec.
against enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses

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MinutenWipes 30

- REF 3050** StarterSet: 2 dispenser boxes, empty + 2 refill bags with 60 wipes each
- REF 3051** carton with 6 refill bags
- MinutenWipes Jumbo 30**
- REF 3052** StarterSet: 2 dispenser boxes, empty + 2 refill bags with 100 wipes each
- REF 3053** carton with 12 refill bags
- REF 4602** **wall holder (round dispenser boxes)** for JumboWipes 30/50

Wipes 30	Wipes 50	Jumbo 30	Jumbo 50	Maxi
14 x 20 cm	14 x 20 cm	21 x 26 cm	21 x 26 cm	17 x 26 cm
30 g/m ²	50 g/m ²	30 g/m ²	50 g/m ²	50 g/m ²
60 Stk./pcs.	40 Stk./pcs.	100 Stk./pcs.	70 Stk./pcs.	250 Stk./pcs.

Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment.

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Active ingredient basis

Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.

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- | | |
|---|--------|
| • virucidal (EN 14476, EN 16777) | 2 min. |
| against non-enveloped adeno and noro viruses as well as all enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses | |

Test criteria: without soiling, room temperature*

- | | |
|---|---------|
| • virucidal (RKI/DVV) | 30 sec. |
| against enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses | |

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Application only on alcohol compatible surfaces by qualified medical personnel!

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MinutenWipes Maxi

REF 4747 StarterSet: 1 dispenser box, empty
+ 1 refill bag with 250 wipes

REF 4714 carton with 4 refill bags

REF 4603 **wall holder (square dispenser box)**
for MaxiWipes

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Alcohol-free solution ready for use for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines particularly with acrylic glass or metal surfaces.

Field of application

Particularly suitable for surfaces which are sensitive to alcohol.

Active ingredient basis

Alcohol-free, aldehyde-free and phenol-free formulation based on sec. and tert. alkylamines.

Spectrum of activity

Test criteria: clean conditions, room temperature*

- | | |
|---------------------------------------|--------|
| • bactericidal (EN 13727, EN 16615**) | 5 min. |
| • yeasticidal (EN 13624, EN 16615**) | 5 min. |
| • tuberculocidal (VAH method 14.1) | 5 min. |
| • virucidal (EN 14476, EN 16777) | 5 min. |
- against all enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses

* Pre-cleaning: in accordance with the recommendation of the RKI (Robert Koch Institute), visibly soiled surfaces should be pre-cleaned.

** with MaxiWipes reel or MaxiWipes-L reel

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Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines particularly with acrylic glass or metal surfaces.

- VAH certified, ready for use, lint-free

Field of application

Particularly suitable for surfaces which are sensitive to alcohol.

Active ingredient basis

Alcohol-free, aldehyde-free and phenol-free formulation based on sec. and tert. alkylamines.

Spectrum of activity

Test criteria: clean conditions, room temperature*

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PlastiSept eco Wipes 30

REF 4743 StarterSet: 2 dispenser boxes, empty + 2 refill bags with 60 wipes each

REF 4742 carton with 6 refill bags

PlastiSept eco JumboWipes 30

REF 4744 StarterSet: 2 dispenser boxes, empty + 2 refill bags with 100 wipes each

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PlastiSept eco MaxiWipes

- REF 4740** StarterSet: 1 dispenser box, empty
+ 1 refill bag with 250 wipes
- REF 4626** carton with 4 refill bags
- REF 4603** **wall holder (square dispenser box)** for MaxiWipes

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Einwirkzeit

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Dry reel made of nonwoven for soaking with MinutenSpray-classic or PlastiSept eco for the disinfection and cleaning of surfaces of medical devices.

Soaked with PlastiSept eco

- bactericidal
- yeasticidal
- tuberculocidal
- virucidal
 - all enveloped viruses incl. corona viruses

Contact time

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Soaked with MinutenSpray-classic

- bactericidal (incl. MRSA)
- yeasticidal
- tuberculocidal
- virucidal
 - enveloped viruses
 - non-enveloped viruses

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Field of application, soaked with PlastiSept eco

Especially suited for surfaces which are sensitive to alcohol such as treatment units (e. g. upholstery, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), and surfaces of acrylic glass.

Field of application, soaked with MinutenSpray-classic

Suited for surfaces which are compatible with alcohol if quick drying and a comprehensive disinfection are required.

Active ingredient basis/Efficacy

- soaked with PlastiSept eco see p. 18
- soaked with MinutenSpray-classic see p. 14

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Please make sure that the dry disposable wipes to be soaked are compatible with the used disinfection solution!

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MaxiWipes reel

REF 4612

StarterSet: 1 dispenser box, empty
+ 1 refill bag with 250 wipes

REF 4613

carton with 4 refill bags with 250 wipes, dry

MaxiWipes-L reel

REF 4741

StarterSet: 1 dispenser box, empty
+ 1 refill bag with 160 wipes

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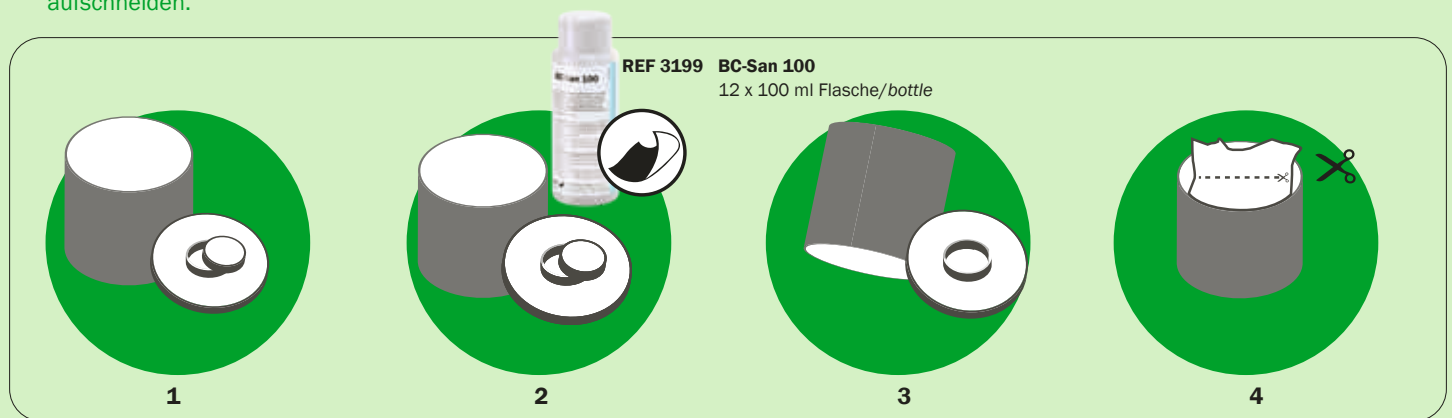
Reprocessing of the dispenser boxes*

- 1 Dispose of solution and wipes beyond expiry date.
- 2 Wipe thoroughly the inside of the empty box and the lid for processing with a wipe soaked with BC-San 100.
- 3 Let box and lid dry.
- 4 Place the bag into the dispenser box and cut it open along the dividing line.

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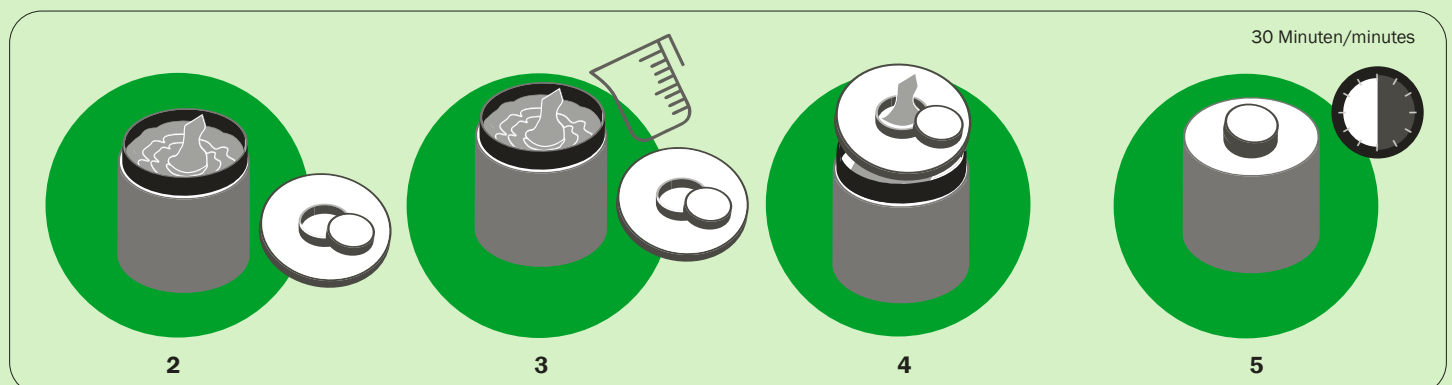
Filling of the MaxiWipes reel dispenser boxes*

- 1 Reprocessing (see above)
- 2 Filling: Put the unpacked reel with bag into the box. Pull out a small piece of the innermost wipe from the middle.
- 3 Soaking: Slowly pour the ready for use solution (MinutenSpray-classic or PlastiSept eco) spirally from the middle outwards over the reel.
- 4 Stick through: Pull the innermost wipe of the reel through the star-shaped tear-off slot of the lid and dispose it. Put the lid on the dispenser and close until it locks.
- 5 Soaking time: Let the wipes rest for 30 minutes.

Manuelle Instrumentenreinigung und -desinfektion Manual cleaning and disinfection of instruments

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BIB forte eco

Liquid concentrate free of aldehydes, phenols and QAC for the non protein fixing cleaning and disinfection of medical and dental instruments, rotating precision instruments and endoscopes. Tested on instruments of company Komet, furthermore approved and recommended by e. g. Busch & Co., Dentsply Maillefer, Hager & Meisinger, Helmut Zepf.

Material compatibility

The extremely high corrosion-inhibiting properties could be improved by the QAC-free formulation, thus instruments sensitive to corrosion get optimal protection.

Even with occasional exceeding of contact time BIB forte eco is absolutely compatible with the materials.

Durability in use

7 days at normal contamination.

Active ingredient basis

Modern combination of alkylamine and biguanide.

Spectrum of activity

Test criteria: dirty conditions

- **bactericidal** (EN 13727, EN 14561)
0.25 % in 15 minutes or
3.0 % in 5 minutes
- **yeasticidal** (EN 13624, EN 14562)
0.25 % in 15 minutes or
3.0 % in 5 minutes
- **tuberculocidal** (EN 14348, EN 14563)
0.5 % in 60 minutes or
2.0 % in 15 minutes

Test criteria: clean conditions

- **virucidal** (RKI/DVV, quantitative suspension test)
against enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses and non-enveloped adeno, noro and parvo viruses
0.5 % in 60 minutes or
1.0 % in 10 minutes
- **tuberculocidal** (EN 14563, ultrasonic bath)
1.0 % in 10 minutes or
2.0 % in 5 minutes

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Non protein fixing solution ready for use for the pre-cleaning of medical instruments. Dissolves residues of blood and other body fluids from medical devices (internal and external surfaces).
Approved amongst others by Planmeca, Sirona, Helmut Zepf.

Active ingredient basis

Complexing agent (EDTA), parabens, biguanides in aqueous solution.

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Manual cleaning and disinfection of instruments

WL-clean

Solution ready for use for the non protein fixing cleaning (internal and external surfaces) prior to disinfection/sterilization of medical and dental hollow part instruments such as turbines, hand pieces and contra-angles, internally cooled instruments and endoscopes. WL-clean is a medical device of class I. The proof of suitability for cleaning has been given according to the series of standards EN ISO 15883 and according to the specifications of the German Society for Sterile Goods Supply (DGSV) – and has been confirmed by expert's reports. Tested and approved amongst others by KaVo, NSK, SciCan, W+H, Morita, Sirona.

Active ingredient basis

Biguanides, PHB-esters.

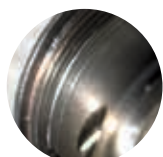
Features

- **cleaning** (practice test with test soiling according to EN 15883-5)
- **complexing**
- **emulsifying**
- **not protein fixing**

before

WL-clean

after

CE

WL-clean
REF 4149

combination set
1 x WL-clean + 1 x WL-dry

REF 4150

clinic package
4 x 500 ml WL-clean

WL-cid

Solution ready for use for the disinfection (internal and external surfaces) of medical and dental hollow part instruments. WL-cid is a medical device of class IIb. The disinfection capacity under simulated practical conditions according to VAH/DGHM has been confirmed by an expert's report.

Active ingredient basis

Aldehyde-free and phenol-free disinfection solution based on a combination of alcohols, QACs and alkylamines.

Spectrum of activity

Test criteria: clean conditions, room temperature, in 5 minutes*

- **bactericidal** (EN 13727, EN 13697)
- **fungicidal** (EN 13624, EN 17387)
- **mycobactericidal** (EN 14348, EN 13697)
- **virucidal** (EN 14476, EN 16777) against enveloped viruses, e.g. HBV, HCV, HIV, influenza as well as non-enveloped viruses, e.g. adeno, noro, polio and polyoma viruses (SV 40)

* **after pre-cleaning with WL-clean**

Important note:

For the correct reprocessing, please refer to our latest SOP.

CE 0123

WL-cid
REF 4141

combination set
1 x WL-cid + 1 x WL-dry

REF 4140

clinic package
4 x 500 ml WL-cid

WL-dry

Medically pure drying spray to support the cleaning after the cleaning and/or disinfection of the inner surfaces of medical and dental hollow part instruments (turbines, hand pieces and contra-angles, internally cooled instruments and endoscopes). WL-dry is a medical device of class I.

WL-dry contains: 1,3,3,3-Tetrafluoroprop-1-ene, (CAS-no.: 29118-24-9)

Field of application

Removing of residues of cleaning and disinfection agents and maintenance products and drying of spray channels and inner surfaces.

Content

100 % chemical inert propellant, meets the hygienic requirements on „medical air“ for the processing of medical hollow part instruments.

Features

- drying
- supporting the cleaning

CE

WL-dry
REF 3165

clinic package
4 x 300 ml

REF 4147

StarterSet
2 x WL-clean, 2 x WL-cid, 2 x WL-dry

WL-Blow

The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of hollow part instruments and transmission instruments after cleaning and disinfection with WL-clean/WL-cid.¹ WL-Blow is intended for use in the processing room and can be connected to the existing oil-free dry air compressor!²

¹ Please note manufacturers' approvals!

² Connection of a bacterial filter is recommended!

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The rinsing device „WaterJet“ with integrated water flow limiter is used to pre-rinse and facilitates the cleaning before the further processing of outer and inner surfaces of dental transmission instruments and general instruments. The rinsing device “WaterJet” is designed for the use in the processing room and can be connected to the drinking water line.

Application only by qualified personnel!

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Rinsing adapter for ultrasonic tips/sonic tips M 3 internal thread from EMS, Mectron, KaVo, Komet.

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Rinsing adapter for ultrasonic tips M 3/a internal thread from Satelec, NSK.

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Rinsing adapter for ultrasonic tips/sonic tips in the ultrasonic surgery external thread from Sirona (Siroprep and Siroimplant).

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Rinsing adapter for ultrasonic tips in the ultrasonic surgery internal thread from Acteon, (2nd generation) e. g. for Piezotome.

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Combined adapter to clean and disinfect surgical contra-angles and handpieces externally cooled with the WL-series. Consisting of the U-1 rinsing adapter for ultrasonic tips and U-1a screw attachment for hose plug connection.

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Universal adapter with Luer connector for transmission instruments (ISO 9168) for bringing-in of cleaning and maintenance products (e. g. WL-clean, WL-cid, maintenance oil) and for blowing-out of with WL-dry/WL-Blow. Also suitable for blowing-out by using the multifunction syringe and to bring in the cleaning solution WL-clean via Luer connection e. g. into other hollow part instruments.

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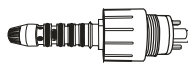
Adapter only for spray water and spray air channels of turbines with Sirona-interface.

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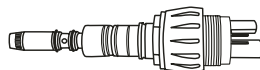
Adapter only for spray water and spray air channels for turbines with Multiflex-(KaVo/SciCan) interface.

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Further adapters for spray water and spray air channels of other turbine manufacturers on request.

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Adapter for **all** contra-angles and hand pieces with ISO-interface for complete internal cleaning/disinfection incl. head of instrument.

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Adapter for turbines with Sirona interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for turbines with Multiflex-(KaVo/SciCan) interface for complete internal cleaning/disinfection incl. head of turbine.

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Introductory set

with 2 WL-adapters*

REF 3541

WL-Adapter 02/G, 04/G inkl. 1 Set Ersatz-O-Ringe.

WL-adapter 02/G, 04/G incl. 1 replacement set of O-rings.

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Adapter for turbines with Bien Air-interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for turbines with Roto-Quick-interface (W&H) for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for longshank contra-angles T1 Classic (Sirona) for complete internal cleaning/disinfection incl. head of instrument.

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Adapter for turbines with NSK-interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for turbines with Morita-interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for ACTEON-AIR N GO Airpolisher Supra/Perio nozzle for complete internal cleaning/disinfection incl. head of instrument.

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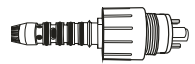
Adapter for KaVo MF handpiece cannulas for complete internal cleaning/disinfection.

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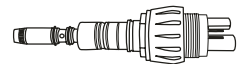
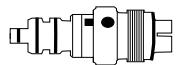
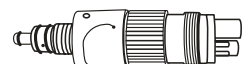
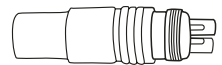
Solution for the virucidal final disinfection of surfaces of medical devices, such as light guides, polymerisation lamps, heads of intra-oral cameras/scanners, as well as for the disinfection/processing of wipe dispenser boxes and pressure bottles (1 – 1.5 L) of stand-alone procedural water supply systems such as the ALPRO-BCS (BottleCareSystem).

Virucidal final disinfection:

After pre-cleaning and general disinfection (e. g. with PlastiSept eco Wipes/MinutenWipes) soak a suitable cellulose wipe with undiluted BC-San 100, thoroughly wipe off the surfaces to be disinfected and let it take effect at least 5 minutes.

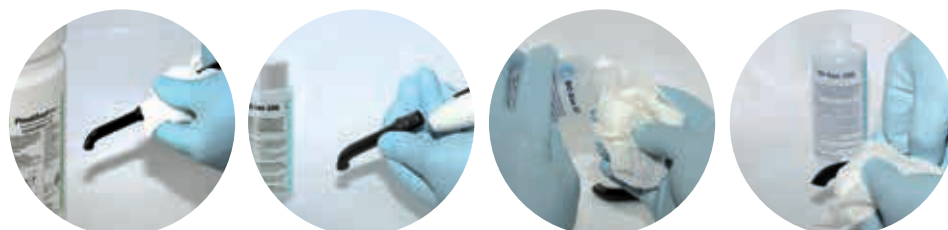
Disinfection effect tested with parvoviruses and *B. subtilis* spores.

Spectrum of activity (see instruction for use)

Test criteria: clean conditions, room temperature

- **bactericidal** in 10 minutes
- **yeastcidal** in 10 minutes
- **sporicidal** in 10 minutes
- **virucidal** in 5 minutes

Note: pre-disinfection with TB effective disinfectant

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CE

Mild alkaline liquid cleaning concentrate for the application in automatic and semi-automatic dosing systems of washer-disinfectors, for the automatic processing of medical devices.

Field of application

- alkaline cleaner (pH 10) for very effective removal of blood and protein residues at all temperature programs
- especially gentle to materials
- protects instruments made of steel and stainless steel from corrosion
- suitable in combination with TR-Neutralizer also in washer-disinfectors such as Miele thermodesinfectors. Approved by Melag.

Active ingredient basis

Optimized cleaning concentrate based on mild alkaline tensides, QAC, complexing agents and phosphonates.

Dosage

dosage (washer/washer-disinfector): 0.3 – 0.5 %

ultrasonic device: 5 – 10 ml/L

Fill the concentrate into the door dosage system for the cleaner and close the flap. For automatic dosage systems connect the dosage lance with machine-compatible can and if necessary adjust the dosage quantity.

Application only by qualified medical personnel!

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**TR-3
REF 3717**

5 L can, compatible for direct connection with dosage lance

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*Liquid concentrate for the neutralization after the alkaline cleaning in washer-disinfectors.
Approved amongst others by the company Miele Professional.*

Field of application

*Neutralization component based on phosphoric acid for use in washers and washer-disinfectors during thermal preparation in the Vario TD program or RKI procedure for neutralization after the alkaline cleaning.
Also suitable for basic cleaning of dental instruments (as immersion bath).*

Active ingredient basis

Universally applicable neutralizing solution for all alkaline cleaning concentrates based on phosphoric acid.

Features

- neutralizing
- prevents corrosion

Application only by qualified medical personnel!

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Inhalt

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Coarse-grained special salt for water softeners in washers and washer-disinfectors.

- optimum rinsing result
- prevents calcium deposits
- gentle on material

Content

Highly purified NaCl, without suspending agent.

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Regeneration powder for automatic cleaning devices.
For the intensive cleaning and preservation of all types of washers/
washer disinfectors (chemical/chemical-thermal). Maintains the
cleaning and disinfection efficiency of the machine.

- powerfully removes deposits of grease, proteins and lime

Active ingredient basis

Highly effective cleaning granulate for the cyclic cleaning of the
inside, lines and sump pit based on sulfamic acid.

The application of RegClean is carried out without load.

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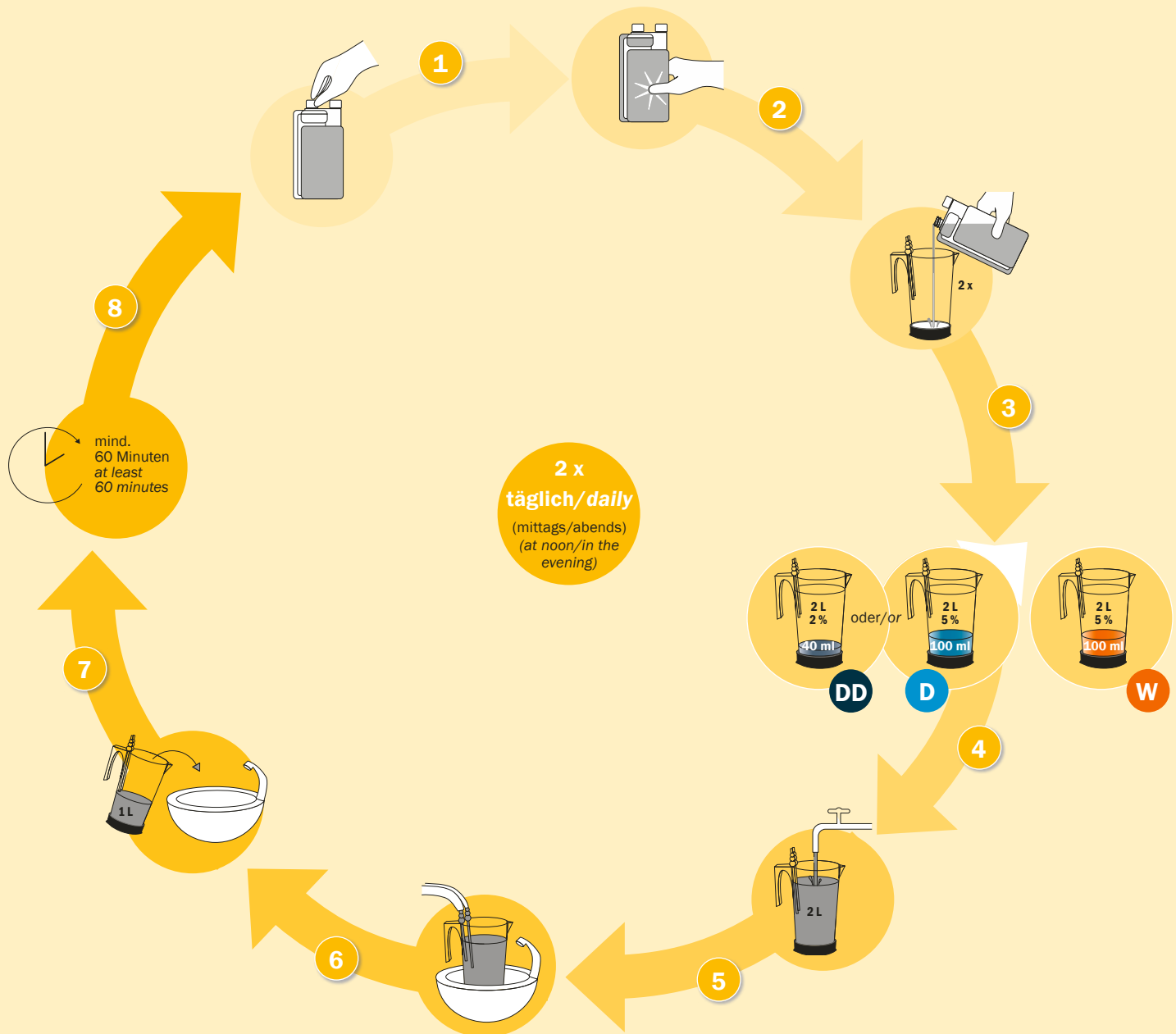
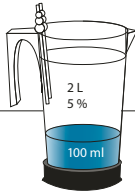
Liquid concentrate for the daily disinfection and simultaneous cleaning of dental aspiration systems with and without amalgam separator.

Application

For reliable disinfection it is recommended to use AlproJet-D for 3 consecutive working days 1 – 2 times each day (e.g. noon, evening). For the remaining 2 working days the application with AlproJet-W is recommended.

Note: Do not use AlproJet-D on invasive products.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning- and disinfection concentrate based on QAC, tensides and alkaline carriers.

Spectrum of activity

Test criteria: dirty conditions, room temperature

- **bactericidal** (EN 13727, EN 14561)
 - 1.5 % in 60 minutes
 - 3.0 % in 30 Minutes
- **yeastcidal** (EN 13624, EN 14562)
 - 1.5 % in 60 minutes
 - 3.0 % in 30 minutes

Material compatibility

Tested and approved for use on dental units by well-known dental equipment manufacturers (including Dentsply Sirona, Morita) with regard to material compatibility. This means that AlproJet-D can be used in the dental unit overnight and/or over the weekend.

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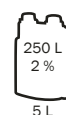
Liquid concentrate with enhanced disinfecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator.

Application

For the reliable cleaning, disinfection and care it is recommended that AlproJet-DD is used on 3 days a week, 1 – 2 times per day (at noon and in the evening) instead of AlproJet-W. For one application 2 L of a 2 % solution are needed. Let take effect for at least 60 min. (EN 16777); rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate with extended disinfection effect based on QAC, amines, tensides and alkaline carriers.

Spectrum of activity

Test criteria: dirty conditions, room temperature

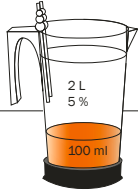
- **bactericidal** (EN 13727, EN 14561) 2 % in 30 minutes
- **yeastcidal** (EN 13624, EN 14562) 2 % in 30 minutes
- **virucidal** (EN 14476, EN 16777) against all enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses 2 % in 60 minutes
- **virucidal** (EN 14476, EN 16777) against non-enveloped adeno and noro viruses as well as all enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses 2 % in 60 minutes

Very good compatibility with materials. Approvals from leading device manufacturers, amongst others by Dentsply Sirona.

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Liquid concentrate for the weekly acid contra-cleaning and maintenance of dental aspiration systems with and without amalgam separator.

Application

For the reliable cleaning and care it is recommended that AlproJet-W is used on 2 days a week, 1 – 2 times per day (at noon and in the evening) instead of AlproJet-D or AlproJet-DD. In case of high protein load AlproJet-W can also be used temporarily every day. For one application 2 L of a 5 % solution are needed. Let take effect for at least 30 minutes; rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate for the acid contra-cleaning based on sulfamide acid, QAC and tensides.

Efficacy

- in combination with AlproJet-D/AlproJet-DD comprehensively cleaning and disinfecting

Compatibility

Very good compatibility with materials.
Approvals from leading device manufacturers, amongst others by Dentsply Sirona.

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Ready-to-use liquid cleaning agent for the spittoon bowls of dental units.

Approved amongst others by Dentsply Sirona.

Application

Spread AlproCleaner generously (10 – 20 ml) in the spittoon bowl and remove visible residues with a plastic brush. Rinse with clear, cold water.

Active ingredient basis

Quick-acting, alkaline cleaning solution for the direct application based on alcohol, QAC, tensides and complexing agents.

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Solution ready for use, free of aldehydes and phenols, for the cleaning and disinfection of all kinds of dental impressions, worn dental prostheses and dental works.

Active ingredient basis

Amines, QAC, alcohol and tensides.

Spectrum of activity

Test criteria: dirty conditions, room temperature

- **bactericidal** (EN 13727, EN 14561)
in 5 minutes
- **yeastcidal** (EN 13624, EN 14562)
in 5 minutes
- **virucidal** (EN 14476) against enveloped viruses, e.g. HBV, HCV, HIV, influenza and corona viruses
in 5 minutes
- **tuberculocidal** (EN 14348, EN 14563)
in 15 minutes

Durability in use

Durability in use 2 days (with normal contamination) respectively when cloudy.

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*Liquid concentrate for the cleaning of impression trays, mixing spatulas and instruments.
Active solution can also be used in ultrasonic devices.*

Active ingredient basis

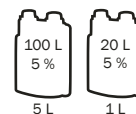
Highly effective, pH-neutral cleaning concentrate for the dissolving of organic and inorganic residues, based on non-ionic and anionic tensides and complexing agents.

Durability in use

Durability in use 2 days.

Note:

AlproSol is also suitable for the removal of alginate adhesive varnishes.

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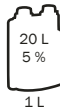
TarClean
REF 3077

1 L dosage bottle

Liquid concentrate for the cleaning and self-acting removal of acid-soluble deposits (e. g. tartar and cement residues) from medical devices such as dental prostheses and instruments.

Field of application

Acidic high-performance concentrate for the cleaning and passivation alloyed medical steels, ideal for cleaning of prostheses.

- *with corrosion inhibitor*
- *also suitable on aluminium*

Active ingredient basis

Phosphoric acid and derivatives, tensides and polyols.

Application

Mix 10 – 20 % with water. Application of 5 % in ultrasonic devices and in case of low soiling. Rinse the cleaned object thoroughly with water afterwards.

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Alkohol und Tenside.

SteriCleaner is used for cleaning and care of inner surfaces/chambers of auto-claves in the dental and medical practice.

Field of application

Quick-acting, alkaline solution for the support of the mechanical cleaning.

- *removal of residues of blood, disinfectant residues (instrument disinfection) and tissue residues*
- *powerfully removes in combination with SteriSponge all residues, which have formed during the sterilization process, even if they are old or heavily burned into material*
- *powerfully cleans with best material compatibility (as it is free from grinding parts)*

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Alpron
REF 3183

1 L Flasche

Liquid concentrate for the continuous maintenance of the water quality in procedural water lines (including instruments and tumbler filler) of medical and dental units.
Approved by notable dental unit manufacturers.

Active ingredient basis

Biguanides, complexing agents and tosylchloramide sodium.

Spectrum of activity:

Test criteria: with soiling, room temperature

- **bactericidal** (EN 13623, test organisms: *Klebsiella pneumoniae*, *Pseudomonas aeruginosa*) in 12 hours
- **conservation** (EuAB 5.1.3, test organisms: *Burkholderia cepacia*, *Enterococcus faecium*, *Legionella pneumophila*, *Pseudomonas aeruginosa*) 1 % for 28 days

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REF 3081

500 ml Messbecher

Solution ready for use for the decontamination and prevention of biofilm formation in procedural water lines of medical and dental units.

Field of application

Solution for the sanitation, conservation and cleaning of the procedural water lines of medical units.

Application of Bilpron at least once to twice a week in the Weekend-System II or ALPRO-BCS as well as other autarkic medical water supply systems removes effectively organic and inorganic residues in procedural water lines (including the instruments and the tumbler filler).

The contact time for the disinfection solution ready for use should be at least 12 hours (overnight) in the procedural water lines, it can also be used as long term disinfection (holidays) up to 3 months.

Active ingredient basis

Propylene glycol, PHB-esters, biguanides and complexing agents in an aqueous solution.

Spectrum of activity:

Test criteria: with soiling, room temperature

- **bactericidal** (EN 13623, test organisms: *Klebsiella pneumoniae*, *Pseudomonas aeruginosa*) in 2 hours
- **conservation** (EuAB 5.1.3, test organisms: *Burkholderia cepacia*, *Enterococcus faecium*, *Legionella pneumophila*, *Pseudomonas aeruginosa*) for 28 days

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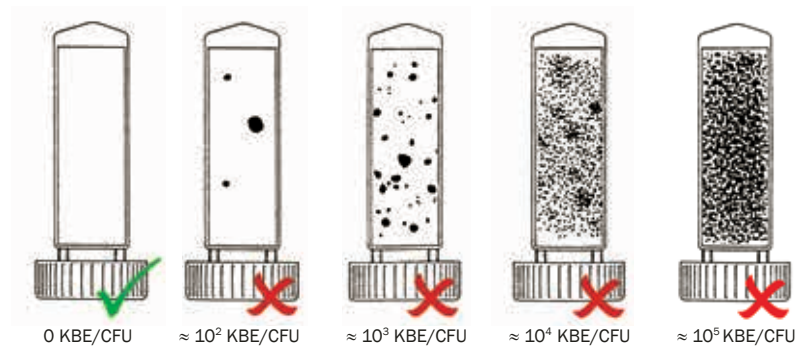
Dip slide for semi-quantitative monitoring of microbial load of water and surfaces.

Field of application

The dip slide is used for the semi quantitative monitoring of the microbial load of liquids and surfaces in industry, environment and medicine.

The microbial count in liquids is measured in "colony-forming units" (CFU) per ml, on surfaces in CFU per cm².

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Biotest
REF 3079
REF 3127

Dip slide (pack of 10)
Biotest evaluation

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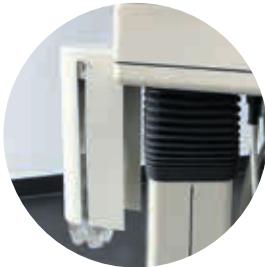
Consisting of BottleCareSystem (BCS) and Bilpron StarterSet.

ALPRO-BCS: Procedural water supply system for medical and dental units.

Bilpron StarterSet: Set for the basic cleaning and disinfection of procedural water lines of medical and dental units.
Meets the requirements according to **DIN 1988-100/EN 1717** and provides an easy hygiene management of the dental unit water lines and the connected dental unit.

Features

- each dental unit can be individually upgraded
- meets the standards DIN 1988-100 and EN 1717
- drinking water with Alpron or drinking water without disinfecting solution can be used for the treatment
- the disinfection of the water lines with **Bilpron** is carried out weekly in idle times (at least for 12 hours) or during vacations

ALPRO-BCS (BottleCareSystem)	
REF 8757	StarterSet (with external housing)
REF 8775	pressure bottle (1.5 L with riser pipe)

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Field of application

Concentrate for the routine processing of ALPRO-BCS (BottleCareSystem).

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Note: *pre-disinfection with TB effective disinfectant*

Application

Pressure bottles of procedural water supply systems:

*Pour the content of the bottle (100 ml) of **BC-San 100** into the empty pressure bottle and fill it with drinking water up to the thread. Leave the solution in the pressure bottle for at least 6 hours (e. g. overnight). During disinfection, keep the filled pressure bottle closed in the processing room.*

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** JIS Z 2801/ISO 22196

Field of application

- sterile filter for terminal adaption at the tap
- protection of immunodeficient or immunocompromised patients against waterborne pathogenic bacteria

Features

- quick and safe installation
- class I medical device
- compact and handy sterile filter
- filter is designed for targeted water jet guide acc. to RKI recommendations*
- filter housing contains a defined bacteriostatic additive**

* RKI recommendations: 'Hygiene requirements for the medical care of immunocompromised patients' - Federal Health Paper 2010 · 53:357-388.

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i3 ONE Sterilfilter (terminal)

REF 3582 carton with 2 filters

REF 3581 **coupling** with aquastop for water taps with external thread

REF 3580 **coupling** with aquastop for water taps with internal thread

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Field of application

- sterile filter (inline) for use on/filtration of procedural water lines, for protection against water-borne, pathogenic germs
- sterile filter for use on cleaning pistols, e.g. for rinsing instruments

Features

- disposable product, up to 50/90* days operating time
- quick and safe installation
- bacteria retention according to ASTM F383-15A
> 7 log per cm² (Brev. dim.)
- high and stable flow rate over the entire period of use

* 90 days when operating the i3 FOUR sterile filter with deionized water
(fully demineralized water) or when operating the i3 FOUR sterile filter with a
low flow rate (< 50 liters per day)

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REF 4603 Wandhalter (eckige Spenderbox) wall holder (square dispenser box)

REF 4690 Deckel, eckig lid, squared

REF 4692 Spenderbox*, eckig dispenser box, squared

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*replacement pump
(for wall dispenser with lever
arm)*

für universelles Spendersystem
for multipurpose dispenser

REF 3519

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Hebelarm, grau (1 L/500 ml)
*wall dispenser with lever arm,
grey (1 L/500 ml)*

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*Multipurpose dispenser for soaps and alco-
holic solutions, drip tray inclusive. Suitable
for 1 L and 500 ml Euro bottles.*

REF 4024

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Hebelarm, weiß (1 L/500 ml)
*wall dispenser with lever arm,
white (1 L/500 ml)*

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*Multipurpose dispenser for soaps and alco-
holic solutions, drip tray inclusive. Suitable
for 1 L and 500 ml Euro bottles.*

REF 3082

proCheck TI 2

Proteinnachweis-Test
Protein Detection Test

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*Set for the determination of residual
protein for 2 instruments*

REF 3083

proCheck TI 3

Proteinnachweis-Test
Protein Detection Test

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*Set for the determination of residual
protein for 3 instruments*

REF 3086

proCheck Endo

Proteinnachweis-Test
Protein Detection Test

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*Set for the determination of residual
protein for 1 endoscope*

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